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The foldable FFP2 respirator mask offers high wearing comfort and effective protection of the respiratory tract against solid and liquid aerosols.

INSTRUCTIONS FOR USE

Before using the mask, the following information must be read and followed. Failure to follow the instructions for use can result in illness or death!

- The user of the mask must be instructed on how to use it correctly.
- Masks do not protect against gases and vapours.
- The oxygen content in the breathing air must be minimum 19.5 % by volume.
- Masks must not be used if the concentration, type or characteristics of the harmful substances are not known.
- The mask must be changed immediately if it is damaged or the breathing resistance increases.
- After the end of a working shift, the mask must always be changed and is not reusable (only to be used for one shift)
- Never make changes or modifications to the mask.

STORAGE AND SHELF-LIFE

Shelf life when properly stored max. 3 years. Check expiry date on masks before use. Storage and transport clean and dry between -5°C to +25°C at max. 60 % humidity, in original packaging. Protect against UV radiation!

DISPOSAL

In case of damage, high pollution or if the breathing resistance is significantly increased, the mask should be changed and disposed in accordance with regional regulations.

INSTRUCTIONS FOR PUTTING ON THE RESPIRATOR

Wash hands before putting on the mask. Prevent contact with the inside of the mask.

Please note: Only stretch the rubber band facewards. Otherwise the ear band fixation may be destroyed.

Take the respirator mask out of the packaging, unfold it and pull the ear band/head band firmly in both directions. Inspect the mask visually for damage.

Place the mask in your hand with the nose area facing up. Now direct the respirator to your face. First insert your chin, then pull the mask over your nose. Fix the side ear bands/head bands.

For a firm fit, it is necessary to adjust the clip in the nose area to the contour of the face.

Checking the tightness: cover the mask with both hands and exhale vigorously. With slight pressure build-up in the mask (exhale), no air should escape at the edges. When used correctly, the filter material moves slightly when inhaling.

Warning: Hair like beards can affect the required firm fit of the mask and can cause leaks.